POLSKI

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Cleaning MOISTURIZING
Protein removing
Rinsing **DISINFECTING**
STORING

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e Rev. 2021/01/04

ENGLISH

Clean Active Aqua plus is an all in one solution with sodium hyaluronate and protein removing agents, ideal for disinfecting, cleaning, storing and hydrating all soft and coloured contact lenses. Thanks to sodium hyaluronate, Clean Active Aqua plus has a unique balanced formulation offering not only effective antibacterial action, but also superior comfort and long lasting hydration.

Contents:

Polyhexamethylene Biguanide 0.0001%; Disodium edetate 0.1%; amphoteric cationic Surfactants 1.001%; made up to 100% with Excipients and Purified water.

Instructions for use:

Wash your hands carefully before using the product. Remove the lenses and place them in the palm of your hand. Add enough Clean Active Aqua plus solution to completely wet the lenses. Gently rub the lenses with your fingers for a few seconds. In order to completely remove all deposits, rinse the lenses thoroughly with CClean Active Aqua plus solution. Repeat the process for the other lens. Fill the lens case with Clean Active Aqua plus solution up to the dosage line. Carefully close the lens case, leaving the lenses immersed in the solution for at least 4 hours or overnight.

Precautions

Discard any unused Clean Active Aqua plus 60 days after first opening the bottle. Do not use the product after the expiry date. Do not use the product if the seal is broken. This product is sterile until opened. If lenses are left in the same Clean Active Aqua plus solution for more than 7 days, repeat the cleaning process with fresh Clean Active Aqua plus. Change the lens case once a month to reduce the risk of contamination.

Indications

Keep the product out of reach of children.

Contraindications

DO NOT INGEST. Do not use tap water to rinse lenses.

Warnings

Carefully follow the instructions written on the label/box.

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ESOFORM S.r.l.
KODANO S.A.